Serenity Choice™ Pro/Plus

EN User Guide

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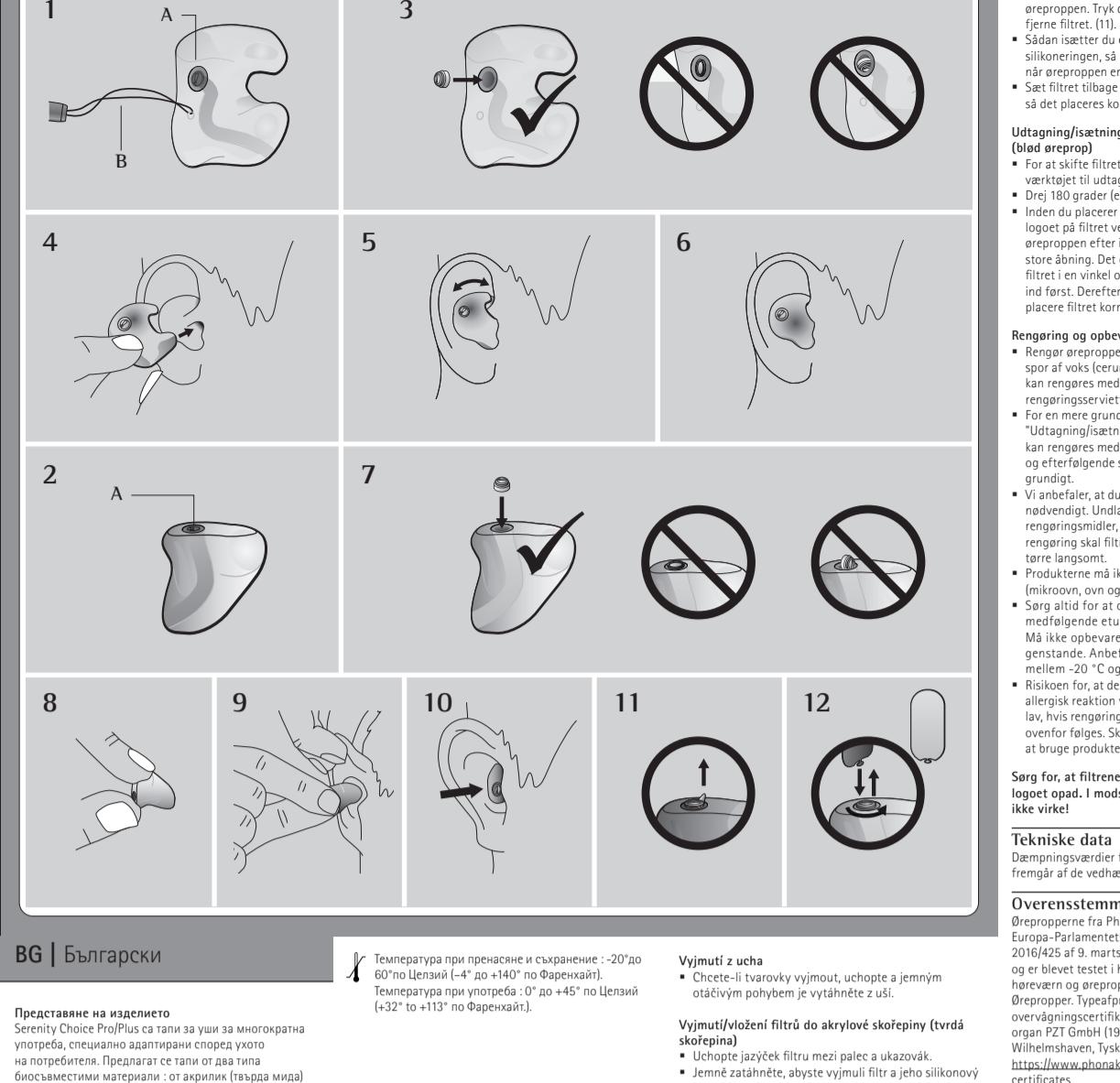
EN | English

Product Overview

Serenity Choice Pro/Plus are re-usable Ear-Plugs, custom-adapted to the user's ear. The Serenity Choice Pro/Plus are made from two different materials: acrylic (hard shell) and soft medical-grade silicone (soft shell) and two different form factors: full contra (11) or mini (2).

Important User Information
• If these instructions do not apply to the product, then follow the instructions of the manufacturer of the product.

• The product is intended to be placed in the ear and to reduce noise levels from noise source levels.

• The product must be fitted, adjusted and maintained in accordance with the instructions for use. Failure to do this may reduce its effectiveness in attenuating noise.

• The material and design of the ear-mold is long-lasting, but the shape of the ear-mold may change over time due to age. Your weight (body weight) significantly. This could adversely affect the sealing of the ear-mold.

• Before inserting the product into the ear canal, first ensure the filter is correctly positioned in the position of the tip (1).

• Please note that the product and your hands are clean before insertion in order to reduce the risk of infection.

• The product may be adversely affected when coming into contact with chemicals or alcohol.

• Choking hazard: Keep the product out of reach of young children.

• Earplugs always have a certain amount of warning sounds, alarms and other important signals. Always pay extra attention to your surroundings.

• Keep the ear-molds clean after use. Remove any water or debris from the ear and form factor.

• The ear-molds are not designed to withstand temperatures below 20°C or above +40°C.

• Suitable hearing protection is recommended at all times in noisy environments. Wearing hearing protection for several hours at a time can cause discomfort. Take a break by removing the hearing protection in a safe environment to reduce strain on your ears.

Instructions for Use

For full-contra (Acrylic and Silicone Insert)
• Hold the ear-mold red for your right ear, the ear-mold black mould for your left ear.

• Insert the ear-mold into the ear canal with the opening pointing into your ear canal (4).

• Twirl the ear-mold slightly to insert it fully into your ear canal. The ear-mold will fit firmly in your ear and forms a good seal.

Removing/Inserting Filters on Acrylic Shell
• Grip the tongue of the filter between thumb and forefinger.

• Push the filter to remove the filter and its silicone ring off the shell, then press on the edge of the ring to remove the filter (1).

• To reinsert the filter, push the filter into the silicone ring, the logo facing upward, so the logo will face on the shell once in place.

From the ear
• To remove the earplugs and twist in a forward motion while pulling gently away from the ear.

Removing/Inserting Filters on Silicone Shell
• Grip the tongue of the filter between thumb and forefinger.

• Push the filter to remove the filter and its silicone ring off the shell, then press on the edge of the ring to remove the filter (1).

• To reinsert the filter, push the filter into the silicone ring, the logo facing upward, so the logo will face on the shell once in place.

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